

DIMENSIONS: 210 X 297

mx^{HEALTH™} mobility
MOBILISING PATIENTS WITH CARE
WHEELCHAIR
INSTRUCTIONS FOR USE

MD **REF** MX79306



Intended use: A mobility aid specifically designed for individuals with physical disabilities, reduced mobility, or medical conditions that prevent them from walking independently. It is used to assist patients in moving safely within home, clinical, or institutional settings. The product is intended for medical and rehabilitative purposes, offering improved mobility, independence, and quality of life to users. This device is classified as a medical assistive device and is not intended for recreational or non-medical use.

FEATURES:

- Plastic hand rim
- Plastic footplate
- Fixed armrest
- Fixed footrest
- 8" pvc castor wheels
- 24" rear wheels
- Foldable
- Total width: 65cm
- Folding width: 23cm
- Seat width: 46cm
- Total height: 87cm
- Total length: 106cm
- Loading weight: 100kg

⚠ WARNING!

READ ALL INSTRUCTIONS AND WARNINGS PRIOR TO ASSEMBLY AND USE. IF YOU DO NOT UNDERSTAND THESE INSTRUCTIONS, CONTACT A HEALTH PROFESSIONAL BEFORE ATTEMPTING TO USE.

Before using the wheelchair, be sure your weight is within the weight capacity of the unit. This wheelchair will require little or no assembly and can be performed without tools.

This product should always be used on a flat and solid surface. Heavy or uneven weight distribution may cause stress to the frame.

For maximum safety, make sure all safety features are explained to you by a qualified therapist, physician, or a certified salesperson. Check all plastic parts often for wear. Please consult with your physician before using this product especially if you are taking any medication. Failure to observe this warning may result in injury.

Ensure that the brake is locked when getting on/off the wheelchair

DO NOT stand on the frame of the wheelchair.

DO NOT use the footplate as a platform. When getting in or out of the wheelchair, make sure that the footplates are in the upward position. ALWAYS use the hand rims for self-propulsion.

DO NOT operate on roads, streets, or highways.

ALWAYS check hand grips for looseness before using the wheelchair. If loose and/or worn, replace IMMEDIATELY.

When assistance to the wheelchair user is required, remember to use good body mechanics. Keep your back straight and bend your knees whenever tilting the wheelchair or traversing curbs, or other impediments. Make sure occupant's hands and body are clear of all pinch points before reclining back.

Use caution, otherwise injury may occur. When learning a new assistance technique, have an experienced assistant help you before attempting it alone.

DO NOT attempt to lift the wheelchair by any removable (detachable) parts. Lifting by means of any removable (detachable) parts of the wheelchair may result in injury to the user or damage to the wheelchair.

DIMENSIONS: 210 X 297

DO NOT traverse, climb, or go down ramps or slopes greater than 9°.
DO NOT attempt to move up or down an incline with a water, ice or oil film.
DO NOT attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm to you or damage to the wheelchair.
Unless otherwise noted, all service and adjustment should be performed while the wheelchair is unoccupied.
Wheelchair users should NOT be transported in vehicles of any kind while in wheelchairs

SAFETY/HANDLING OF WHEELCHAIRS:


Safety and handling of the wheelchair requires the close attention of the wheelchair user as well as the assistant. This manual points out the most common procedures and techniques involved in the safe operation and maintenance of the wheelchair. It is important to practice and master these safe techniques until you are comfortable in manoeuvring around the frequently encountered architectural barriers. Use this information only as a basic guide. The techniques that are discussed on the following pages have been used successfully by many. Individual wheelchair users often develop skills to deal with daily living activities that may differ from those described in this manual. However, all warnings and cautions given in this manual **MUST** be heeded. Techniques in this manual are a starting point for the new wheelchair user and assistant with "safety" as the most important consideration for all. Be aware that carrying heavy objects on your lap while occupying the wheelchair may adversely affect the stability of the wheelchair, resulting in serious bodily injury to the user, damage to the wheelchair and surrounding property. This wheelchair has been designed to accommodate one individual. If more than one individual occupies the wheelchair this may adversely affect the stability of the wheelchair, resulting in serious bodily injury to the user, damage to the wheelchair and surrounding property

ASSEMBLY:


Inspect all parts for shipping damages before use. If there is shipping damage—DO NOT USE. Contact your dealer/distributor immediately for further instructions. If there are any problems with your product, DO NOT attempt to repair it yourself. Contact your dealer/distributor that you purchased the product from for any needed parts or repairs.
Mx Health assumes no responsibilities for any damage or injury caused by improper installation, assembly, or use of this product.

OPERATION:


Stand behind the wheelchair and pull it outwards to open the chair.
Next, press the seat down to open it.
ALWAYS keep hands and fingers clear of moving parts to avoid injury.
DO NOT place hand or fingers on the underside of the seat frame rail when opening or closing the wheelchair.
DO NOT sit or transfer into the wheelchair unless it is fully open.

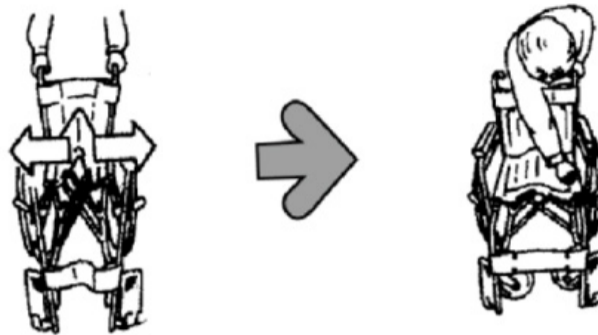


Ensure that the brake is locked when getting on/off the

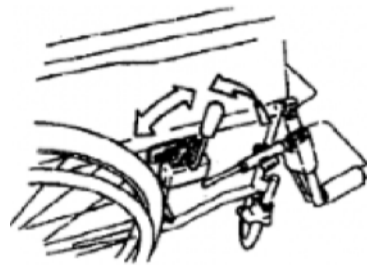

Flip the footplate up whenever getting on or off the wheelchair. Don't step on the foot plate when getting off the wheelchair will tip forward.

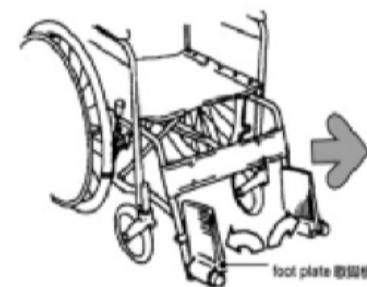

Ensure that footplate is at least 7cm above the ground. If placed too low, the footplate will hit obstacles when travelling on uneven ground, causing the wheelchair to be off-balance.

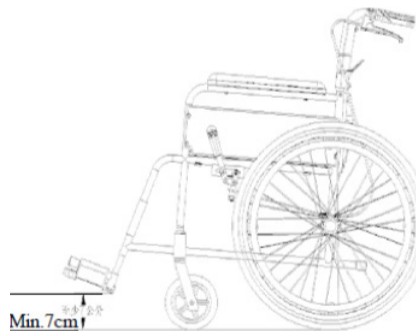


To go up the stairs, the caregiver should push the wheelchair and step on the tipping bar and push downwards at the handles, so that the front castors are raised onto the steps. Next, pull upwards at the handles to bring the rear wheels up. Don't try to push on the steps by force.

To move up or down a steep slope, ensure that a caregiver is around to support the wheelchair, otherwise, the wheelchair may overturn. For a steep downward slope, the wheelchair should be turned to face up slope, and the caregiver should back the chair downwards gradually.



DIMENSIONS: 210 X 297

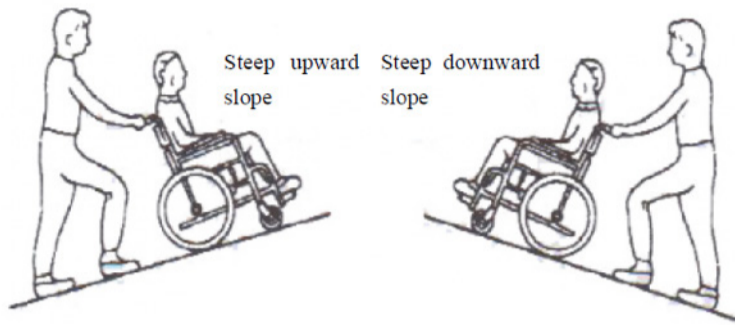

REACHING, LEANING AND BENDING FORWARD OR SIDWAYS:

DO NOT attempt to reach objects if you must move forward in the seat or pick them up from the floor by reaching down between your knees.

DO NOT attempt to reach or lift objects that would cause you to shift weight to one side of the wheelchair. Reaching or lifting objects on the side of the wheelchair can cause the chair to tip over sideways.

DO NOT lean over the top of the back upholstery to reach objects behind you, as this may cause the wheelchair to tip over.


Forward:

Position the front casters so that they are extended as far forward as possible and engage wheel locks.

Backwards:


Position wheelchair as close as possible to the desired object.

Point front casters forward to create the longest possible wheelbase. Reach back only as far as your arm will extend without changing your sitting position.

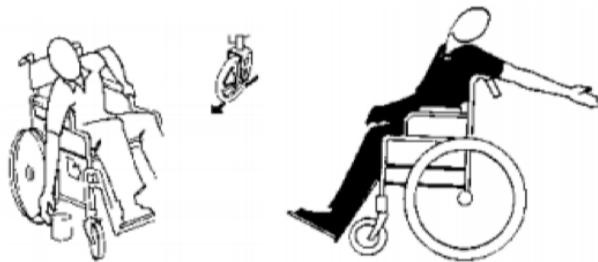

COPING WITH EVERYDAY OBSTACLES:

Coping with the irritation of everyday obstacles can be alleviated somewhat by learning how to manage your wheelchair. Keep in mind your centre of gravity to maintain stability and balance.

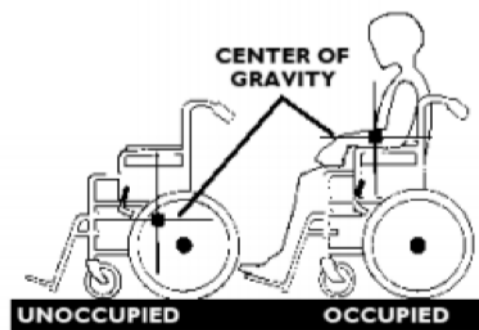

ESCALATORS:

DO NOT use an escalator to move a wheelchair between floors. Serious bodily injury may occur.

MAINTANANCE:

The WHEELCHAIR is designed for minimum maintenance.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water.

Do not use harsh abrasives.

Clean regularly and dry.

When cleaning hand grip areas use only a clean towel lightly dampened with cool water. Verify that grips are dry prior to use.

If the wheelchair is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure that the handgrips DO NOT twist on the handle.

Otherwise, damage or injury may occur.

Always make sure your brakes are operating correctly before use. Keep the brake handles free from dirt, dust, and grime.

Build-up of particles could cause your brakes not to engage properly.

If the brakes are not braking, check the cables for looseness. The cables can be tightened at the adjuster screw located where the cable is connected to the brake handle.

If the cable is broken, it will need to be replaced. Periodically check and tighten all nuts, bolts, screws, and other hardware on WHEELCHAIR.

After any adjustments, repair, or service and before use, make sure all attaching hardware is tightened securely. Otherwise, injury or damage may result.

DO NOT overtighten hardware attaching to the frame. This could cause damage to the frame tubing


As with any vehicle, check the wheels and tyres periodically for cracks and wear. Replace if damaged.

Device readiness: With the device fully assembled as per the instructions, and checked for safety, the device is ready for use.

Reusability: If the device becomes worn, or becomes dilapidated, discontinue use, and obtain a new device.

When to consult a healthcare professional: If the user complains about any discomfort or inconvenience using the device, consult with a healthcare professional.

⚠ Should any problems be experienced from the use of this device, please notify us immediately at customercare@medinox.co.uk. Should any serious incident or harm be the result of the use of this device, please inform the manufacturer and the health authorities in the country of sale.

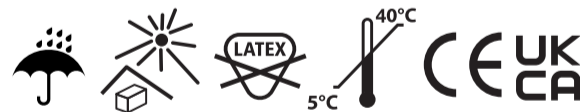

UK Manufacturer: Medinox (London) Ltd. Sterling House, Fulborne Road, London, E17 4EE, United Kingdom. www.medinox.co.uk

South African Applicant: Medinox (Pty) Ltd, 2 Sunrock Close, Sunnyrock, Germiston, Gauteng, 1401. www.medinox.co.za

Australian Sponsor: Medinox Australia (Pty) Ltd, 1/10 Albert Ave, Broadbeach, QLD, 4218. www.medinox.com.au

New Zealand Responsible person: Medinox Australia (Pty) Ltd, Suite 13732, Level 1, 6 Johnsonville Road, Johnsonville, Wellington 6037, New Zealand.

Medinox Health Ireland Ltd, Ground Floor, 71 Lower Baggot Street, Dublin D2, D02 P593 Ireland.



Publication date: March 2026

